

PROVIZIO[®]

SEM SCANNER GATEWAY



Introducing Provizio[®] SEM Scanner, the first hand-held wireless scanner with intelligence inside, for fast and easy Pressure Injury/Ulcer (PI/PU) risk assessments and the only SEM Scanner that acts as an adjunct to routine clinical skin assessment.

The device scans and collects patient data in real-time, site-by-site and patient-by-patient. When placed in the Provizio[®] Charging Hub, the collected data automatically transfers to the comprehensive Gateway Dashboard for patient, ward and facility management purposes.



Seamless data transfer and management

Capture and transfer patient level Provizio[®] SEM Scanner data on a select patient or wider patient population seamlessly and directly to the Gateway Dashboard. Health Care Practitioners (HCPs) across the care pathway can review and share anatomically-specific data to help identify PI/PU risks for all their patients, regardless of skin tone.¹ All data is collected, encrypted and stored in digital format, on the hospital's internal server in the Gateway Dashboard.

The Provizio[®] SEM Scanner Gateway Dashboard allows the user to:

- View patient deltas Δ for individual patient assessments
- +
- Track a patient's Δ from admission and at time points; by day, week, month, year or at change in condition
- +
- View active Provizio[®] SEM Scanners and total number of Δ
- +
- View patient, setting and total population Provizio[®] SEM Scanner data

Provizio® SEM Scanner Gateway Dashboard

	Action	Time	BBI Role	Facility IT Role
Setting up the Gateway	1 Provizio® SEM Scanner Wi-Fi set-up	5-10 minutes per device	Support Facility IT to set-up Wi-Fi connection for each device	Either; Provide BBI with the Wi-Fi password and log-in details and BBI will set-up all devices or Facility IT to set up the devices by using the Provizio® SEM Scanner Gateway Support Manual
	2 Install Provizio® SEM Scanner Gateway Executable File	2 hours (1 person)	Provide Executable File. Install if BBI have approved access to facility server - can be any server inside the firewall	Facility IT runs Executable File as per BBI instructions provided
Using the Gateway	Log in and view Provizio® SEM Scanner Gateway data	2-5 minutes per patient	Provide the Provizio® SEM Scanner User Manual which lists details of how to log-in and view patient level data	No action
	Maintenance of Provizio® SEM Scanner Gateway	4-5 hours annually (1 person)	BBI to provide flash updates as available	Facility IT loads flash updates to the devices on a pull basis

EHR/EMR Integration

	Action	BBI Role	Facility IT Role
Integrating into EHR/EMR	Match patient ID to patient name in the Facility electronic medical records	No action	Pull patient name and match to scan patient ID (Note - common practice for medical devices ie. ultrasound and medication management)
	EHR/EMR integration	Provide template for data layout	Determine template for data layout Facility IT liaises with EHR/EMR provider to load updated template to the EHR/EMR system

BBI will provide the following:

Provizio® SEM Scanner;

- Device, Charging Hub and Sensors
- User Manual
- Quick Start Guide
- Product Specification Sheet

Provizio® SEM Scanner Gateway;

- Executable Software (USB)
- User Manual
- System Support Manual

BBI Europe Ltd. - Phone: +44 (0)1625 238 895
Bruin Biometrics LLC - Phone: +1 310 268 9494

info@bruinbiometrics.com
www.sem-scanner.com
www.bruinbiometrics.com



To find out more scan this code or visit our website

www.sem-scanner.com